

MAXIFILM

FAST DRYING SURFACE TENSION REDUCING AGENT

APPLICATION

Maxifilm is a surfactant solution formulated to gently reduce the surface tension of impressions, prior to pouring models, and of wax patterns, prior to investing. Treating impressions and wax patterns with Maxifilm also aids in the removal of dust and oily residues accumulated over time. Together, these features allow for better flow and adaptation of the dental material being poured, virtually eliminating the formation of bubbles and voids on the surfaces of models and castings. Maxifilm is suitable for use under vacuum and is available in refillable fine-mist pump spray bottles.

DIRECTIONS

1. Remove gross debris from impression / wax pattern.
2. Apply sparingly with gentle brush or fine-mist pump spray bottle.
3. Remove excess surfactant by gently shaking.
4. Allow impression / wax pattern to dry.
5. Pour gypsum / investment under gentle vibration.

CAUTION!

This product contains alcohol. Use mask and avoid inhalation. Keep closed after use. Read instructions before use.

EMDIN INTERNATIONAL CORPORATION
15841 Business Center Drive, Irwindale, CA 91706
Tel: (626) 813-3740 Fax: (626) 445-5233
E-mail: info@emdin.com Website: www.emdin.com

EMDIN[™]
INTERNATIONAL CORPORATION

© Emdin International Corporation, Rev. 9/2010